biological Calprotectin Card

Rapid Test to detect
human calprotectin in
card format

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biological health
ready to test

ISO 13485 CERTIFIED
MEDICAL DEVICE:
QUALITY MANAGEMENT SYSTEM

biotical Calprotectin card

ENGLISH

WHAT IS BIOTICAL CALPROTECTIN CARD?

biotical Calprotectin card is a colored chromatographic immunoassay test (qualitative cut off test) for the detection of human Calprotectin (hCp) in stool samples. Thanks to its easy handling and high sensitivity, the healthcare professional will be able to make a presumptive diagnosis of Inflammatory Bowel Diseases (IBD) as well as a follow up of the treatment and to predict the risk of relapse.

GASTROINTESTINAL INFLAMMATION AND CALPROTECTIN

Human Calprotectin (hCp) is a calcium-associated protein that has bacteriostatic and fungicidal properties. In inflammatory processes their plasma levels rise considerably, with high levels of human Calprotectin being found in the faeces of patients with various inflammatory bowel diseases (IBD), such as Crohn's disease and ulcerative colitis. This fecal human Calprotectin assay is useful for differentiating an Inflammatory Bowel Disease (IBD) from functional Irritable Bowel Syndrome (IBS). Human Calprotectin, once released until its degradation, remains stable at ambient temperature for 7 days in the feces, thus being an ideal analyte. This is because it is extremely resistant to proteolytic degradation (proteolysis).

biotical Calprotectin card is a simple, non-invasive test, easy to use and with almost immediate results, based on the detection of human Calprotectin in stool samples. This serves to distinguish patients with inflammation (invasive bacterial infection, IBD, etc.) from those without inflammation (viral, toxigenic, etc.). In addition, the determination of fecal human Calprotectin concentrations may be useful for predicting imminent clinical recurrence in various IBD. Considering that children need general anesthesia for colonoscopies, the benefits will be even greater.

HOW DOES BIOTICAL CALPROTECTIN CARD WORK?

The card contains inside a test strip with a nitrocellulose membrane that has been previously pre-coated with:

- **In the test line (T):** with mouse monoclonal antibodies against human Calprotectin (hCp).
- **In the control line (C):** with rabbit polyclonal antibodies against a specific protein.

Sample area (S): the pad where the sample is placed has been dispensed with a preparation of reagents of the T line (mouse monoclonal antibodies against human Calprotectin) conjugated with latex of red polystyrene; and a control solution (specific binding protein) conjugated to green polystyrene latex, forming two conjugate colored complexes.

TEST LINE (T):

- **If the sample is positive:** The antigens of the patient's sample will react with the red colored conjugate complex (anti-human Calprotectin monoclonal antibodies red latex microspheres for the line T) and advance by capillarity through the membrane. The anti-human Calprotectin antibodies present in the T line will capture the colored test complex resulting in the red line.

- **If the sample is negative:** If the sample is negative and therefore does not contain antigens (or the antigen is present in a concentration below the detection limit), there is no reaction with the red colored complex. The anti- human Calprotectin antibodies present in the T line will not capture the red colored antigen-complex (not formed) and the red line will not appear.

CONTROL LINE (C):

Regardless of whether the sample is positive or not, to verify that the test has been performed correctly (sufficient sample volume, appropriate flow, internal control of the reagents, etc.) the mixture will continue moving through the membrane towards the immobilized antibodies against the specific protein located in the control line (C). These specific anti-protein antibodies present in the membrane will capture the control conjugate complex and the green control line will always appear.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30 °C. The test is stable until through the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contaminations errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contains preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provided in the kit are approved for use in the biotical Calprotectin test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.
- Pale yellow or orange bands may appear in the test strip results window, at the positions where the colored test and control lines should appear, before the test is performed. These bands are completely normal and correspond to the capture antibodies of the test. The existence of these bands does not alter the state of conservation and functionality of the kit.

MATERIALS

MATERIALS PROVIDED

- *biotical Calprotectin card* tests
- Instructions for use
- Stool collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED

- Specimen collection container
- Disposable gloves
- Timer

biological Calprotectin card


Rapid test to detect *human calprotectin* in card format

SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 7 days prior to testing. For longer storage, maximum 6 months, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool samples as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):

- Take out the cap of the stool collection tube and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (1), to collect faecal sample and add it to the stool collection tube. For liquid samples, add approx. 15µL in the stool collection tube using a micropipette.
- Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (2). The stool collection vial with diluted sample can be stored for 7 days in the refrigerator (2-8 °C) prior to testing.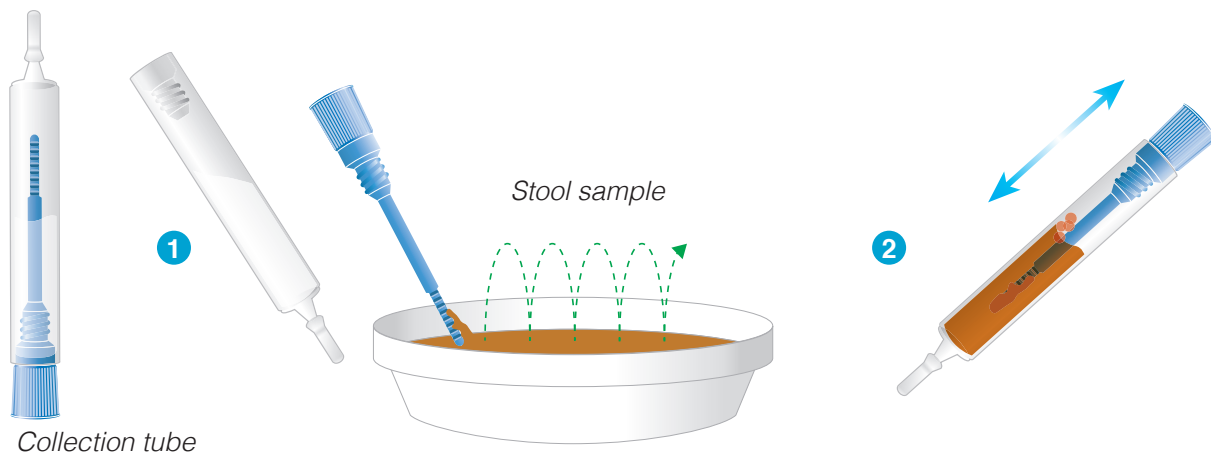


TEST PROCEDURE

Allow tests, samples and controls to reach room temperature (15-30 °C) prior to testing. Do not open pouches until the performance of the assay.


- Proceed to shake the stool collection tube in order to assure good sample dispersion.
- Remove the *biological Calprotectin card* test from its sealed bag just before using it.
- Take the stool collection tube, cut the end of the cap (3) and dispense 4 drops in the circular window marked with the letter S (4). Avoid adding solid particles with the liquid.
- **Read the results at 10 minutes. Do not read the test result later than 10 minutes.**

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.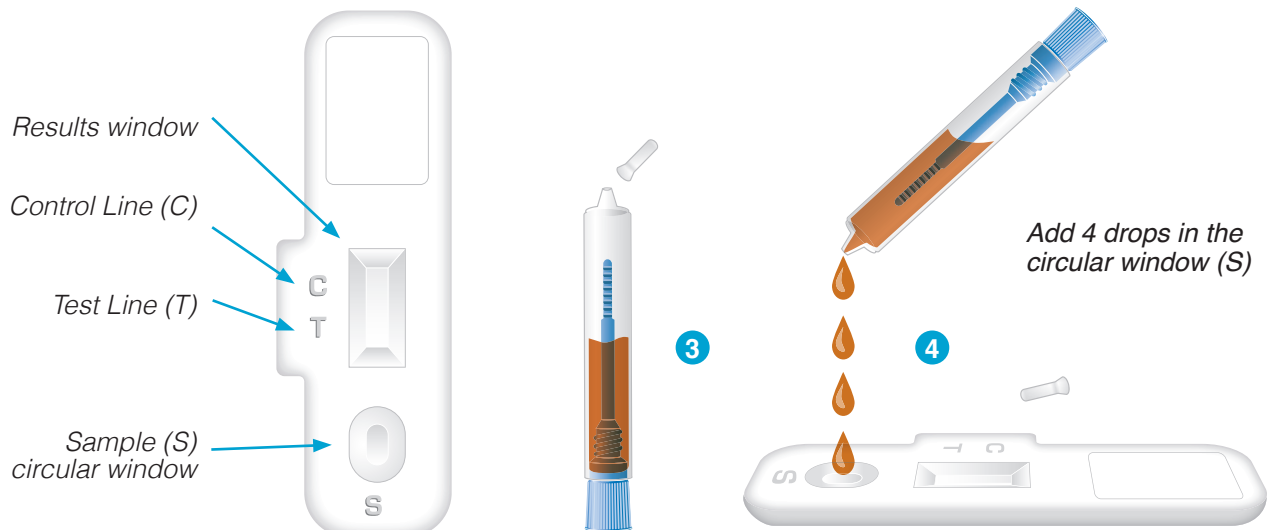

INTERPRETATION OF RESULTS

(please refer to the illustration below)

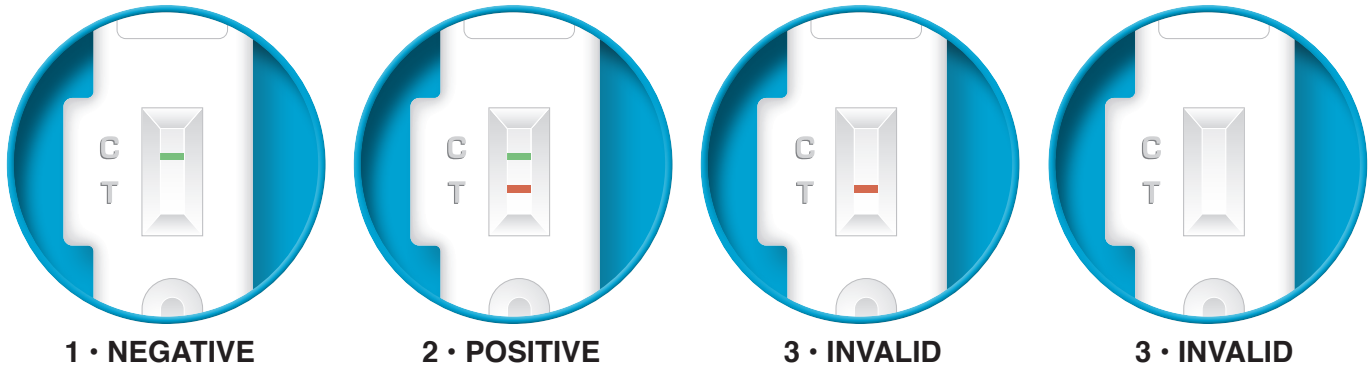	biotical test	Interpretation of the results
1	- GREEN	There is no human calprotectin presence which might mean neither active gastrointestinal inflammation, nor risk of relapse (CD or UC relapse).
2	+ GREEN - RED	There is human calprotectin present which might mean active gastrointestinal inflammation or risk of relapse in clinical remission.
3	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

- **NEGATIVE:** Only the GREEN line appears across the results window in the site marked with the letter C (control line).
- **POSITIVE:** In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.
- **INVALID:** Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

LIMITATIONS

- The test must be carried out within 2 hours after opening the sealed bag.
- An excess of stool sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
- The intensity of test line may vary from very strong at high human calprotectin concentration to faint when the human calprotectin concentration is close to the cut-off value of the test.
- biotical Calprotectin should be used only with samples from human faeces. The use of other samples

biotical Calprotectin card

Rapid test to detect *human calprotectin* in card format

has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.

- Positive results determine the presence of human calprotectin in faecal samples; nevertheless, it can be due to several causes (Inflammatory bowel disease, colorectal cancer and some other enteropathies). A positive result should be followed up with additional diagnostic invasive procedures, a colonoscopy and a biopsy in order to confirm the diagnosis and to establish the inflammation extent.
- A negative result is not meaningful because of it is possible the human calprotectin concentration in the stool sample is lower than the cut-off value. If symptoms or situation still persist, calprotectin determination should be carried out invasive techniques. Negative results do not exclude inflammation, some diseases such as celiac sprue and microscopic colitis polyps that mainly involve mononuclear inflammation.
- Stool samples from patients with non-steroidal anti-inflammatory drug (NSAID) treatment could show positive result.
- Neonatal faecal calprotectin levels have been reported higher than those in normal children with a mean of 167µg/g (range 22-860µg/g).
- Stool from patients with active inflammatory bowel disease that usually involve significant neutrophilic inflammation of the intestine, such as Crohn's disease and ulcerative colitis, would be positive for faecal calprotectin (the level of calprotectin in Crohn's disease is higher than in ulcerative colitis) and would be positive for faecal lactoferrin. biotical Calprotectin could be sensitive for this diagnosis in patients with chronic diarrhea.

EXPECTED VALUES

Some studies established equal or higher 50µg hCp/g faeces as cut-off value to allow detect adult patients with gastrointestinal inflammatory pathology that will require to diagnosis additional diagnostic invasive procedures.

PERFORMANCE CHARACTERISTICS

Analytical sensitivity

Cut-off value of biotical Calprotectin is 500 ng/mL (50 µg hCp/g faeces) for human calprotectin

Clinical sensitivity and specificity

An evaluation was performed comparing an immunochromatographic assay (biotical Calprotectin, biotical) and another commercial immunoassay (Calprest®, Eurospital).

The results were as follows:

		ELISA test: Calprest®		
		+	-	Total
IC test: biotical Calprotectin card	+	34	2	36
	-	2	26	28
	Total	36	28	64

	Sensitivity	Specificity	PPV	NPV
biotical Calprotectin card vs Calprest®	> 94%	93%	> 94%	93%

The results showed a high sensitivity and specificity to detect human calprotectin using biotical Calprotectin.

CROSS REACTIVITY

An evaluation was performed to determine the cross reactivity of biotical Calprotectin; no cross reactivity against other faecal markers occasionally present in faeces:

Bovine and pig haemoglobin	Human haemoglobin
Bovine and pig transferrin	Human transferrin
Bovine lactoferrin	Human lactoferrin

REFERENCES

1. Angriman I. et al. *Enzymes in feces: Useful markers of chronic inflammatory bowel disease. Clinica Chimica Acta* 381 Feb 2007, p. 63-68.
2. Quail, M.A. et al. *Fecal Calprotectin Complements Routine Laboratory Investigations in Diagnosing Childhood Inflammatory Bowel Disease. Inflamm Bowel Dis, Vol 15 No 5; May 2009, p. 756-759.*
3. Gaya D.R., et al. *Faecal calprotectin in the assessment of Crohn's disease activity. Q J Med* 2005, Vol 98, May 2005, p. 435-441.
4. Langhorst, M.D. et al. *Noninvasive Markers in the Assessment of Intestinal Inflammation in Inflammatory Bowel Diseases: Performance of Fecal Lactoferrin, Calprotectin and PMN-Elastase, CRP, and Clinical Indices. Am. J. Gastroenterol.* 2008; Vol 103, p. 162-169.

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