


# biotical health

IN VITRO DIAGNOSTIC DEVICES  
ready to test

## Strep A Card


Rapid Test to detect  
*Group A Streptococcus*  
in human throat swab specimens

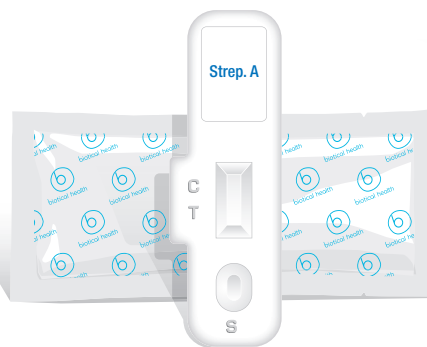

**QUICK, ACCURATE AND EASY-TO-USE TEST**

	Sensitivity	Specificity	PPV	NPV
<b>biotical Strep A</b> vs. OSOM® Strep A Test	> 99%	> 99%	> 99%	> 99%

RESPIRATORY PATHOGEN LINE

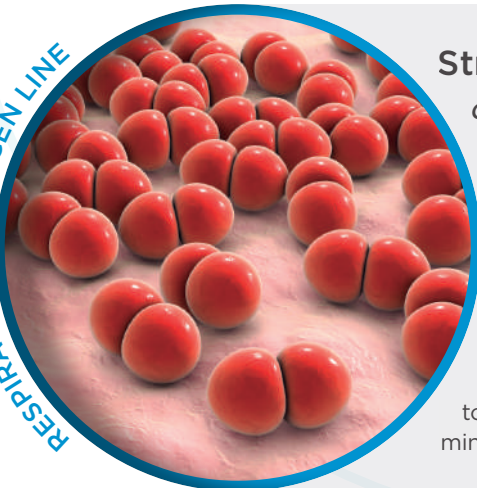

### Streptococcus A

*Group A Streptococcus (Strep)* is a bacterium that is commonly found in the throat and causes diseases such as tonsillitis and pharyngitis. Symptoms of this infection include fever, stomach pain and inflammation and redness in the tonsils, and may be moderate or severe. The treatment in case of infection by Group A Streptococcus is the taking of antibiotics, so an early detection helps to treat the infection before and get a faster cure. Similarly, confusion with infections of viral origin is avoided for which the taking of antibiotics is not prescribed facilitating the rational use of them.

*biotical Strep A card* is an immunochromatographic qualitative test, easy to use and with almost immediate results, for the detection of Group A Streptococcus directly in nasopharyngeal swab samples, providing results in several minutes.


Ref.:  
RTB25SC



PRODUCT MANUFACTURED AND DEVELOPED IN SPAIN

## Strep A card Test procedure

1

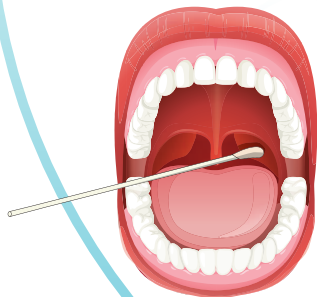

Remove the swab from its packing and collect specimen with a sterile swab from the tonsils and/or the back of the throat

2

- Add 5 drops of Reagent A.
- Add 5 drops of Reagent B.
- Mix the solution with the swab at least 1 minute.




3

Add 4 drops in the circular window "S"




10'


Read the results after 10 minutes

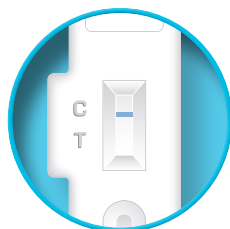


NEGATIVE

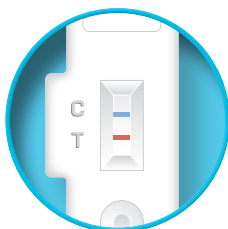

POSITIVE

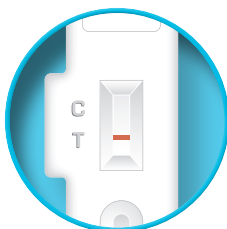

INVALID

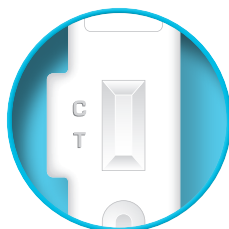

INVALID

<b>biotical Strep A kit</b>	Test card in sealed bag	Swab in sterile package	Test tubes and pipettes	Reagents	Extra quality control
Kit contents	25 pcs.	25 pcs.	25 + 25 pcs.	Included	Positive Control included