


# biotical health

IN VITRO DIAGNOSTIC DEVICES  
ready to test

## Influenza A+B card


Rapid Test to detect  
*Influenza A and B*  
from nasal swab

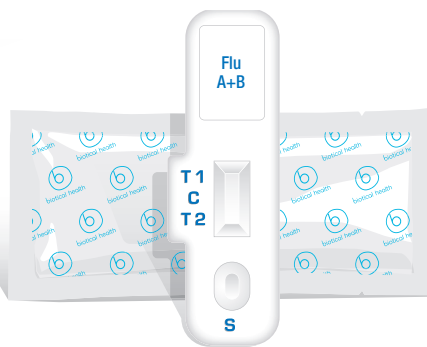

**QUICK, ACCURATE AND EASY-TO-USE TEST**

	Sensitivity	Specificity	PPV	NPV
biotical Influenza A+B card vs BinaxNow® Influenza A&B	> 99%	> 99%	> 94%	> 99%

RESPIRATORY PATHOGEN LINE

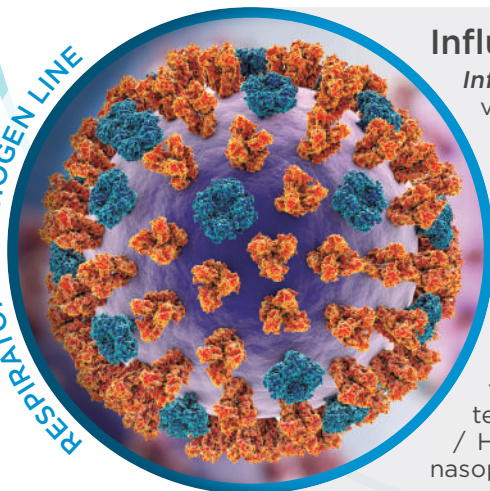

### Influenza virus

**Influenza** is a highly contagious respiratory disease caused by Influenza viruses (or Influenzaviruses) that generally infect the upper respiratory tract, throat, and bronchi. Sometimes, although less commonly, it also affects the lungs. Although the infection is usually overcome in a week, there are certain populations that are at a higher risk for developing complications (children, the elderly, pregnant women, immunosuppressed).

A quick and accurate diagnosis means reducing the risk of transmission and outbreak, preventing inappropriate use of antibiotics and ensuring early prescription of antiviral medication for high-risk patients.

**biotical Influenza A+B card** is an immunochromatographic qualitative test, easy to use and with almost immediate results, for the detection of Influenza virus A subtypes (including subtypes A / H1N1, A / H3N2, A / H5N1, etc.) e Influenza virus B directly from the patient's nasopharyngeal sample.


Ref.:  
RTB25  
ABC

PRODUCT MANUFACTURED AND DEVELOPED IN SPAIN

## Influenza A+B card

Test procedure. Nasal swab method.

- Collect specimen with a sterile swab from one nostril. Repeat procedure using other nostril.
- Process the swab as soon as possible after collecting the specimen.

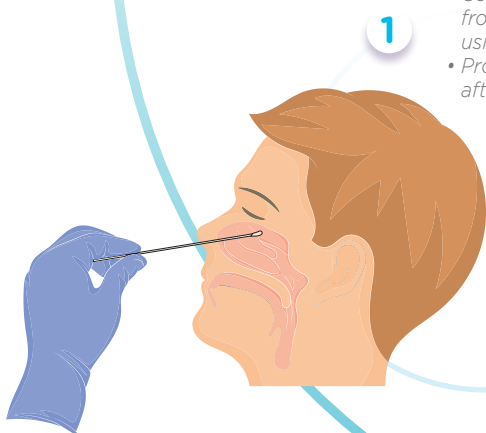


1



2

Add 15 drops of Reagent B.  
Put the swab into the tube, rotating 1 minute and extract the liquid

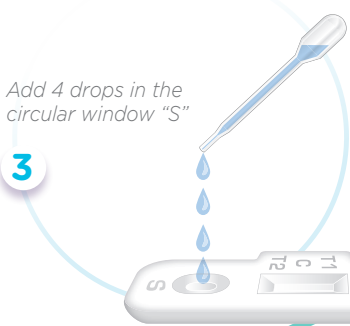


3


Add 4 drops in the circular window "S"

10'


Read the results after 10 minutes

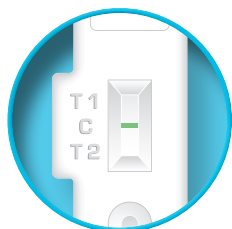


NEGATIVE

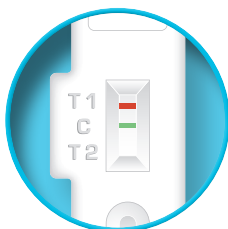


FLU A POSITIVE  
FLU B NEGATIVE

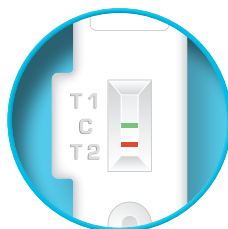

FLU A NEGATIVE  
FLU B POSITIVE

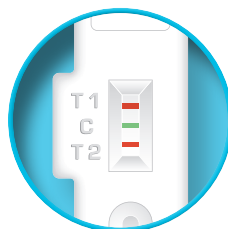

FLU A POSITIVE  
FLU B POSITIVE

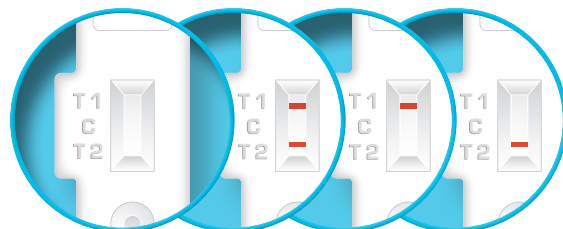

INVALID RESULTS

<b>biotical Strep A kit</b>	Test card in sealed bag	Swab in sterile package	Test tubes and pipettes	Reagents	Extra quality control
Kit contents	25 pcs.	25 pcs.	25 + 25 pcs.	Included	Positive Control included